

## 3-D Laserscanning Anti-Glare Spray and Cleaner



### 3-D Anti-Glare Spray

Art. No.: 119.990.001

PU: 12 cans / box

### 3-D Cleaner

Art. No.: 119.990.002

PU: 12 cans / box

To obtain satisfactory test results surfaces of objects must be prepared with anti-reflective coatings before scanning by the 3-D laser scanning technique.

**HELLING 3-D Laser Scanning Spray** is very fine-grained. Therefore, it creates a thin white and homogeneous anti-reflection coating on the surface of the object.

The surface to be scanned can be prepared optimally with the **HELLING 3-D Laser Scanning Cleaner**.

#### Application:

- Clean surface of object to be scanned with 3-D Laser Scanning Cleaner before measuring
- Shake Anti-Glare Spray well before using
- Spray Anti-Glare Spray thinly on the surface from approx. 20 cm distance
- After scanning the Anti-Glare Spray can be removed easily with the 3-D Cleaner

Heidgraben, December 2022

Subject to technical changes